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User Guide

Foldable Portable Massage Table

Congratulations on the purchase of this portable massage table. Your new table has been skillfully crafted to high standards, utilizing high quality components and materials. With proper use and care, as outlined in this user's guide, your table should provide a longtime of reliable performance. To ensure the safe and correct use of your table, please take a moment to read this entire guide prior to using your table.

PRECAUTIONARY INSTRUCTIONS

- (1) Maintain the table periodically. Make sure there are no cracks in the massage table.
- (2) Do not overload the table beyond the recommended weight limit. This could result in serious injury to the user.
- (3) Do not use alcohol based cleaners when cleaning the vinyl. This cause damage to the vinyl material. Use warm soapy water and wipe dry.
- (4) Make sure when using the table that all four legs are on solid flat surface and at the same level.


Table set up

Place the table on its side,
release the buckles,
partially open the table ,
and remove the accessories(Figure 1)

Figure 1

Standing behind the table ,
rasp both table handles and gently
swing in an outward motion to fully open
the table. The legs should open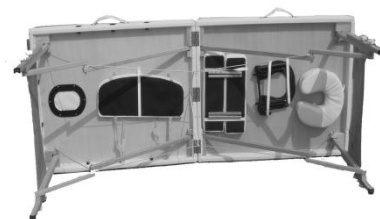

automatically (FIGURE 2)

Turn the table upright by lifting the middle of the table. Press down in the middle of the table to make sure the table lays flat. If the table peaks in the middle, check and straighten out the cables.

Table and the accessories use

We provide optional accessories for you. Please read instruction carefully before uses.

Headrest:

The headrest consists of a crescent headrest base and a crescent face pad. It will give your client comfortable head and face support.

you may have different option for the headrest base. The adjustable headrest is easy for you to change the angle and height of the base by adjusting the base handle (FIGURE 3)

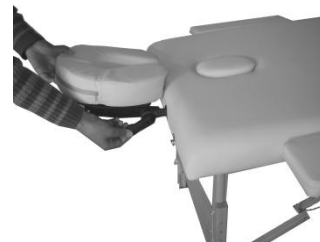
Figure 3

Adjustable armrest shelf:

This adjustable armrest shelf is a good device support your client's arms at the front of the table. Hang the armrest shelf on the two bars of the headrest and slide the belt up or down for the optimum position (FIGURE 4)

Figure 4

Arm sling:

This is an alternative armrest accessory with the same function of the armrest shelf. Please refer to the usage notes of the armrest shelf

Figure 5

FIGURE 5

Side arm extension:

This is the side arm extensions option for our table. These removable padded side armrests add the table width. They fit into the required accessory holes in each side of the table. That will give a comfortable arm support especially for bigger clients.

Height adjustment:

Proper table height is determined by the therapist height and technique. You can adjust the height by removing the grip knob from each leg. Replace the knobs after adjusting each leg to the desired position.


Table weight limits:

Position the client in the center of the table, this will help distribute the weight evenly and ensure stability. Gently sit near (not on) the center of the table at first then sit as close to the center (from the edge) as possible the working weight is the combined weight of the client and the maximum amount of downward pressure being applied by the therapist. For example for a client weighing 136kg, the maximum working weight for the table is 226kg, the maximum downward pressure that can safely be applied is 90kg, as pressure applied at sideward angle puts considerably more stress on a table, the maximum sideward pressure should not exceed 50% of the maximum downward pressure.

DO NOT EXCEED REFERRED MAXIMUM WORKING

WEIGHT (Never Exceed The Load Limits Of The Table;

Doing So May Result In Serious Injury And/Or Damage

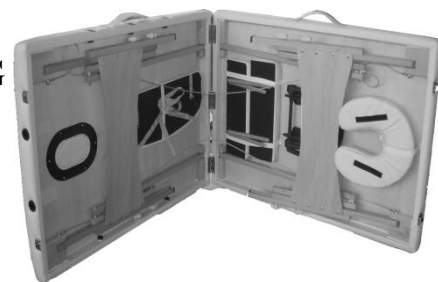**Storage:**

Attach headrest base and face pad onto the elastic under the table as shown. Put all

accessories at their original position.

The carry case should be used when transporting or storing the table to protect the table from damage. When not in use, store the table in a cool dry place, and away from direct sunlight. Do not expose the table to extreme (hot or cold) temperatures or moisture.

Table care

Upholstery care

Daily cleaning: remove daily dirt and grime from the table with a mild soap and warm water solution and dry with a soft lint-free cloth.

Disinfecting: Disinfect the table as required, with an isopropyl alcohol based hospital grade disinfectant cleaner (approved for use on polyurethane vinyl).

Stain removal: some minor stains can be removed with a mild non-abrasive cleaner. Spray a damp cloth with cleaners and gently wipe to remove stain, rinse thoroughly with warm water and dry with a soft lint-free cloth. More stubborn stains may require professional cleaning.

Note: never use harsh or abrasive cleaners or chemicals to clean the table. Doing so may damage the upholstery. Avoid using any protectants, as they may harm the vinyl.